Escalon Medical Corp. | 2440 South 179th Street | New Berlin, WI 53146 | USA
800-486-4848 | 262-821-9182 | www.sonomedescalon.com

Viscous Fluid Cannula

REF TR9849-51

Instructions For Use [ENG]

Indication for Use

This single-use device is to be used as an aid for infusing fluid into the posterior chamber of the eye, and is particularly designed for use with high-viscosity fluids. The cannula set is qualified for use at operating pressures of up to 70 psi.

Directions for Use

Circulating Nurse:

- 1) Peel open outer pouch, letting inner pouch drop onto sterile field. Use caution not to touch inner pouch.

Scrub Nurse:

- 2) Connect female luer to desired infusion line or syringe. Twist luer firmly to ensure cannula is tightly secured.
- 3) Open infusion line or activate viscous fluid injection device to test flow.
- 4) Secure cannula to the eye.
- 5) The cannula is now ready for use.

Note: All connections should be checked before use.

Warnings

- 1) This sterile disposable medical device should not be reused. Known risks associated with reuse of device include cross-contamination between patients and mechanical failure of product during use.
- 2) After use, all parts of this disposable pack should be properly disposed.
- 3) If any component in this pack is received in a defective condition, do not use and immediately notify Sonomed Escalon.
- 4) Product contains phthalates (DEHP) within the tubing.
- 5) There are no known specific risks for use of this device with children or with pregnant or nursing women.

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