



Sonomed Escalon™

Irrigating Contact Lens

REF TR9840-11 Flat Lens

REF TR9841-11 Minus Lens

Indication For Use

Assists in visualization of surgical field while providing means for irrigation.

Directions For Use

Circulating Nurse:

1. Peel open the outer pouch letting the inner pouch drop on to the sterile field. Use caution not to touch the inner pouch.

Scrub Nurse:

1. Peel open the sterile pouch, place irrigating contact lens onto the sterile field by turning tray up-side-down.
2. Break and remove paper tape from lens steel tube.
3. Connect female adaptor fitting of tubing to male adaptor end of irrigation solution.
4. Run irrigation solution through contact lens until all air bubbles are clear of the lens.
5. The irrigating contact lens is now ready for use.

WARNING

- This sterile disposable medical device should not be reused. After use, all parts of this disposable pack should be properly disposed of.
- If any component in this pack is received in a defective condition, do not use and notify Sonomed Escalon



Sonomed Escalon™

Escalon Medical Corp. | 2440 South 179th Street | New Berlin, WI 53146

800-486-4848 | 262-821-9182 | www.sonomedescalon.com



STERILE EO

5539 Rev. F (ECO101029)

CM