




Iris Suture Device

REF TR9921

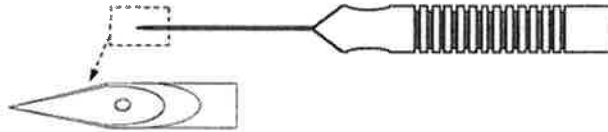

Circulating Nurse

1. Peel open the outer pouch letting the inner pouch drop on to the sterile field. Use caution not to touch the inner pouch.

Scrub Nurse

1. Peel open the sterile pouch, place iris suture device onto the sterile field by turning tray up-side-down.
2. The iris suture device is now ready for use.

Directions for Use

1. Prior to surgery, the fixation needle is passed through clear cornea across the iris to exit anterior to the iris root.
2. A 7.0, 8.0 or 9.0 suture is then passed through the hole in the needle bevel.
3. The needle and suture is then brought into the anterior chamber and passed under the iris out of the eye and tied. Three clock hours may be accomplished with only one initial entrance. The fourth would be placed at a separate entrance site. This instrument is designed for both vitrectomy or posterior and anterior intraocular lens procedures.

WARNING

1. This sterile disposable medical device should not be reused. After use, this product should be treated as potentially biohazardous and should be properly disposed of.
2. If any portion of this product is received in a defective condition (including packaging), do not use and notify Escalon immediately.



Escalon Medical Corp. | 2440 South 179th Street | New Berlin, WI 53146

800-486-4848 | 262-821-9182 | www.sonomedescalon.com

STERILE **EO**

5506 Rev. D (ECO101029)